


pH	Meaning
5.0 to 7.0	Normal
7.0 to 8.0	Normal - add .005 to SG result
8.0	Caution*

Specific Gravity	Hydration Status
1.000 to 1.005	Excellent **
1.005 to 1.010	Very Good
1.010 to 1.015	Good
1.015 to 1.020	Fair
1.020 to 1.025	Caution - Borderline
>1.025	Poor - Unsatisfactory

100 Strips

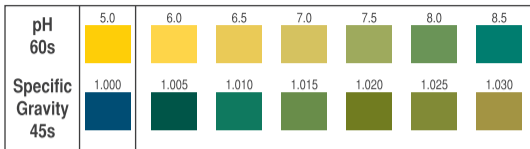

IVD



* A pH of 8.0 may indicate a urinary tract infection or an old sample. Retest with a fresh sample. If the pH is still 8.0 seek medical advice.

** With persistent low readings, **approaching an SG of 1.000**, seek medical advice as it is necessary to rule out over hydration or dilutional hyponatremia.

Distributed by

LFA First Response

Australian helpline 1800 681 544

www.lfafirstresponse.com.au