

REUSABLE MEDICAL FACE SHIELD

CODE 664-182



Call us on 1800 681 544 or visit www.lfafirstresponse.com.au

REUSABLE MEDICAL FACE SHIELD

CODE 664-182

OVERVIEW

Westlab has developed this face shield for the COVID-19 pandemic. The core benefit of the reusable shield is to conserve raw materials and get quality shields to the front line.

- ARTG Registered 333131
- PETG Material (Polyethylene Terephthalate Glycol).
- Foamless Design – Reusable.
- Ships Flat Packed (Easy user assembly).
- Superior Chemical Resistance.
- Full Face Protection.
- One Size Fits All.
- Secure Fit
- Can be used together with an N95 mask and glasses if required

CLEANING

After first use, the outside of protective eyewear/face shields may be contaminated.

Step 1: Remove eyewear/face shield by tilting the head forward and lifting the head band or ear pieces. Avoid touching the front surface of the eyewear/face shield.

Step 2: Obtain Clinell Universal wipes, Quaternary Ammonium Compound or Sodium Hypochlorite Solution 10,000ppm (Or equivalent approved covid-19 sanitisers).

Step 3: Using the wipe/solution, start with the clear shield first then work through to the visor and straps, ensuring full coverage of face shield.

Step 4: Using Put shield aside and leave until dry before use.

This is a cleaning guide that should be completed between shifts by the user, to ensure it is sterile and ready for the next day
wipe/solution, start with the clear shield first then work through to the visor and straps, ensuring full coverage of face shield.



Call us on 1800 681 544 or visit www.lfafirstresponse.com.au

REUSABLE MEDICAL FACE SHIELD

CODE 664-182

PRODUCT IMAGES



Call us on 1800 681 544 or visit www.lfafirstresponse.com.au