

Instructions for Use D95 P2 Respirator Mask



The D95 P2 Respirator is designed to achieve a comfortable and adequate seal by joining the mask's two bands with the FitClip™. Available in two sizes, the FitClip™ allows flexibility in the adequate fitting of respirators. The FitClip™ offers a secure hold point on one end, and an open end for ease of attaching and removing the second mask band.

The Detmold Medical range of D95 P2 Respirators have the Regular Fitclip™ already attached, and where a longer band is required, the Large FitClip™ can be ordered separately.

An inadequate seal between the Respirator and the face can result in contaminated air entering the mask, please refer to your appropriate Respirator Fit Test Procedure to determine the required combination of mask and FitClip™.



Applying the Mask

Prior to donning, ensure hand hygiene is undertaken.



1. Move the Fitclip™ to the band on your preferred side for donning.
2. Open the mask, flattening the nose piece.



3. Lean chin into the mask and lift bands toward the crown of the head, securing onto the Fitclip™.
4. Position the mask nosepiece above the nasal flares, the mask should sit on the soft area of the nose rather than on the bone of the nose.
5. Adjust mask bands for a secure and comfortable fit.



6. Firmly place one finger on the centre of the nose wire, then apply your thumb and first finger to either side of the nose, pinch in along the nosepiece and onto the face.
Note: A very firm pinch around the nose is required to achieve a seal, avoid creating a "peak" in the nosepiece.
7. Once the mask is firmly moulded to the nose, squeeze the nosepiece and gently drop the draw allowing the mask to fit the natural shape of the face.



8. Look left to right, then up and down, there is to be no movement of the mask at the nose wire.
9. Inhale and exhale deeply checking for escape of air.

Note: If there is any movement of the mask at the nose wire or escape of air, repeat Step 6.

Removing the Mask Safely

Prior to doffing, ensure hand hygiene is undertaken.



1. Without touching the mask, follow bands towards FitClip™ and gently unhook, keeping tension on the bands.
2. Lean forward maintaining control of the band tension, whilst slowly removing the mask away from the face. Ensure the mask does not flick forward or back onto the face.



3. Discard safely.

Ensure hand hygiene is performed once the mask has been discarded.

Caution: Mask helps protect against certain particles and is designed for occupational use by healthcare professionals who are properly trained in mask use and limitations. Misuse may result in sickness or death. DO NOT use in any manner not indicated in the Instructions for Use. Always read the label. Discard the mask if it becomes damp, soiled, damaged, or difficult to breathe through. Performance efficiencies may deteriorate when exposed to oil-based substances or individual fit characteristics are not adequate. Do not use if the packaging is damaged. Maximum operating temperature is +50°C.



Call us on 1800 681 544 or visit
www.lfafirstresponse.com.au